

DISINFECTANTS

AMITY PAA 5 SYSTEM

PERACETIC ACID BASED HIGH LEVEL DISINFECTANT / STERILANT FOR FLEXIBLE ENDOSCOPES & MEDICAL DEVICES

Amity PAA 5 is a powerful peracetic acid based high level disinfectant / sterilant concentrate for use in Automated Endoscope Reprocessors (AERs) and other automated decontamination systems. It is an acidic disinfection concentrate for flexible endoscopes and medical devices.

Amity PAA 5 Solutions A and B are combined to produce a single use solution for HLD (High Level Disinfection) / Sterilization of flexible endoscopes with Peracetic Acid.

Amity PAA 5 System are registered as Class IIB Medical Device



Primary features attributable to Amity PAA 5 System are.

- Active ingredient: Peracetic Acid (PAA).
- HLD claim: 5 minutes at 35°C (10 minutes at 20°C).
- Dilution Required: 20ml/L of parts A&B in RO water.
- Contains Corrosion Inhibitors
- Added Water Softeners.
- Tested to EN 14885 for disinfectant properties.
- Tested to EN 15883-4.

Packaging

- 5 litre containers



Amity PAA 5 System - A Peracetic Acid based High Level Disinfectant for Flexible Endoscopes & Medical Devices

DISINFECTANTS

Application

For disinfection of Medical Instruments and Endoscopic Equipment

Stage 1:

Wash medical device carefully in an enzymatic cleaner (Viruzyme Family or Virudet N) and rinse.

Stage 2:

Amity PAA 5 System parts A and B should be dosed together at 20ml/L of each at 35°C (10 mins at 20°C).

Reaction time is 5 minutes in AER.

Rinse with water.

Phase	Time	Temp.	Solution	Dosage
Wash	5 min	25-55 °C	Viruzyme	5ml/L (0.5%)
Rinse	3 min	-	-	-
Disinfection	5 min	35 °C	Amity PAA 5 Solutions A & B	20ml/L (2.0%) of each
	10 min	20 °C		
Rinse	3 min	25 °C	-	-
Rinse	3 min	25 °C	-	-
Dry	-	-	-	-

Drying time and temp according to the machine manufacturer

AMITY PAA 5 SYSTEM

PERACETIC ACID BASED HIGH LEVEL DISINFECTANT / STERILANT
FOR FLEXIBLE ENDOSCOPES & MEDICAL DEVICES

Amity PAA 5 System Properties

Active Against	Standards	Contact Time
Spores	EN17126	5 Minutes
	EN13704	5 Minutes
	EN14347	5 Minutes
Mycobacteria	EN14348	5 Minutes
	Mycobacterium terrae	5 Minutes
	Mycobacterium avium	5 Minutes
	EN14563	5 Minutes
Mycobacterium terrae	5 Minutes	
	Mycobacterium avium	5 Minutes
Bacteria	EN13727	5 Minutes
	EN14561	5 Minutes
Viruses	EN 14476	5 Minutes
	Polio & Adeno	5 Minutes
	HIV, Hep B & C	5 Minutes
Fungi / Yeasts	EN13624	5 Minutes
	EN14562	5 Minutes

Active ingredients: Peracetic Acid, Hydrogen peroxide.



Amity Limited, Libra House, West Street,
Worsbrough Dale, Barnsley, South Yorkshire,
S70 5PG, UK

T: +44 (0) 1226 770787

F: +44 (0) 1226 770757

E: sales@amityinternational.com

WWW.AIHC.CO.UK

© Amity Limited all rights reserved 2023

DISTRIBUTED BY: